

Candidate:	Date:	Evaluator(s):
<b>MINIMUM PREREQUISITES</b>	<ul style="list-style-type: none"> <li>Initial ZOLL familiarization</li> <li>Use of the ZOLL on at least FIVE (5) patients in the field</li> </ul>	
<b>Grading Guideline</b>	<ul style="list-style-type: none"> <li>Candidate must successfully complete ALL <b>RED</b> items in under 10 seconds</li> <li>For all other items, if the candidate can not perform the task within 30 seconds, it is considered a failure</li> <li>Candidate must successfully complete the minimum number of tasks in each category to pass the category</li> <li>Candidate must successfully pass each category</li> </ul>	
<b>/8</b>	<b>OVERVIEW AND POWER-UP (MINIMUM OF 6 NEEDED)</b>	
	<b>Demonstrate how to power-on the ZOLL X Series</b>	
	Locate Quick Access buttons and state their purpose	
	Locate Navigation keys and state their purpose	
	Locate the patient type and show how to change this. <ul style="list-style-type: none"> <li>What does changing this do?</li> </ul>	
	Identify ZOLL integrated communication solutions (Wifi)	
	<b>Demonstrate how to install and remove the battery</b> <ul style="list-style-type: none"> <li>How long do you have to replace the battery before you lose your patient information?</li> </ul>	
	Demonstrate how to use the AC adapter	
	Demonstrate how to change the printer paper <ul style="list-style-type: none"> <li>When should you change the printer paper?</li> </ul>	
<b>/15</b>	<b>GENERAL PATIENT MONITORING (MINIMUM OF 11 NEEDED)</b>	
	Locate the current top lead displayed and demonstrate how to change it	
	<b>Locate heart rate, blood pressure, Respiratory Rate, and spO2</b>	
	Demonstrate how to change automatic NIBP frequency to q15 minutes	
	Demonstrate how to change NIBP to be manually obtained only	
	<b>Demonstrate how to activate and deactivate etCO2</b>	
	Demonstrate how to change the contrast from normal to bright light	
	Demonstrate how to print a strip and stop printing a strip <ul style="list-style-type: none"> <li>How long will this continue to print a strip?</li> </ul>	
	Demonstrate how to take a snap shot <ul style="list-style-type: none"> <li>What does this feature do?</li> </ul>	
	Demonstrate how to pull up the Trends screen and get back to the Home screen	
	<b>Demonstrate how to obtain a 12-lead</b>	
	<b>Demonstrate how to transmit a 12-lead</b>	
	Demonstrate how to review a previous 12-lead	
	Demonstrate how to exit the 12-lead menu	
	Demonstrate all ways to silence alarms	
	Demonstrate how to print a call log	

<b>/5</b>	<b>AED MODE (MINIMUM OF 3 NEEDED)</b>
	<b>Place hands-free pads on patient in proper position</b>
	<b>Demonstrate how to put monitor in AED mode</b>
	<b>Demonstrate how to defibrillate</b>
	Demonstrate how to dump a charge
	Locate where the number of defibrillations is recorded
<b>/2</b>	<b>MANUAL DEFIBRILLATION (PARAMEDIC ONLY) (MINIMUM OF 2 NEEDED)</b>
	<b>Demonstrate how to select appropriate energy</b>
	<b>Demonstrate how to manually defibrillate</b>
<b>/3</b>	<b>SYNCHRONIZED CARIOVERSION (PARAMEDIC ONLY) (MINIMUM OF 2 NEEDED)</b>
	<b>Demonstrate how to activate synchronized cardioversion mode</b>
	Demonstrate how to select appropriate energy level
	<b>Demonstrate how to cardiovert</b>
<b>/6</b>	<b>EXTERNAL CARDIAC PACING (PARAMEDIC ONLY) (MINIMUM OF 4 NEEDED)</b>
	<b>Demonstrate how to turn on Pacer Mode</b>
	Demonstrate how to adjust the Mode <ul style="list-style-type: none"> <li>• When is each mode appropriate?</li> </ul>
	<b>Demonstrate how to adjust the rate</b>
	<b>Demonstrate how to adjust the electrical output</b> <ul style="list-style-type: none"> <li>• How do you determine the patient is being successfully paced?</li> </ul>
	Discuss the "pause" function
	<b>Demonstrate how to turn off the pacer</b>

Candidate has  successfully /  unsuccessfully completed the ZOLL monitor test.

Candidate: \_\_\_\_\_ FTO: \_\_\_\_\_